

Example of a dialysis connection set MA-991-ZEST-487.

1




Before starting the procedure, wash your hands (the specialist and the patient).

2



To perform the procedure, prepare a disinfectant, a tourniquet, suitable cannulas and sets the start and end the procedure.

3

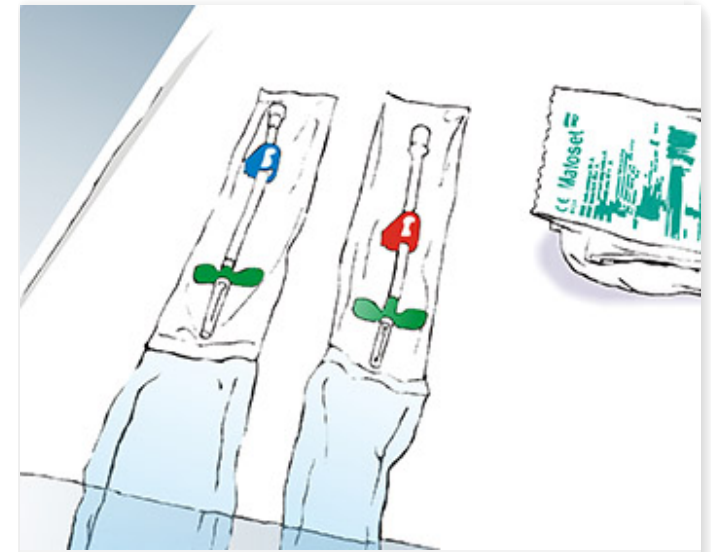


Take out the sterile drape and place it beneath the patient's forearm.

Matoset dialysis connection set - application way

4



5

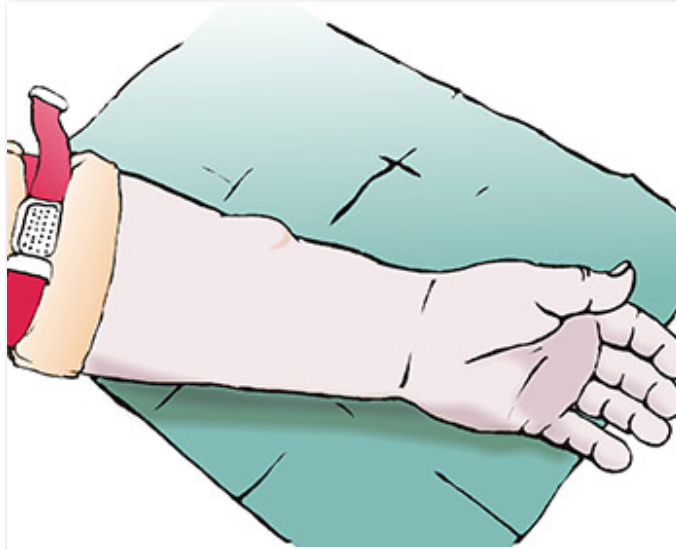

Then put one glove on the left hand, and the other one put on the sterile part of the opened packaging.

6



Matoset dialysis connection set - application way

7



Disinfect the patient's forearm using the prepared disinfection liquid and the sterile swabs from the set.

8



>>


9



>>

Matoset dialysis connection set - application way

10

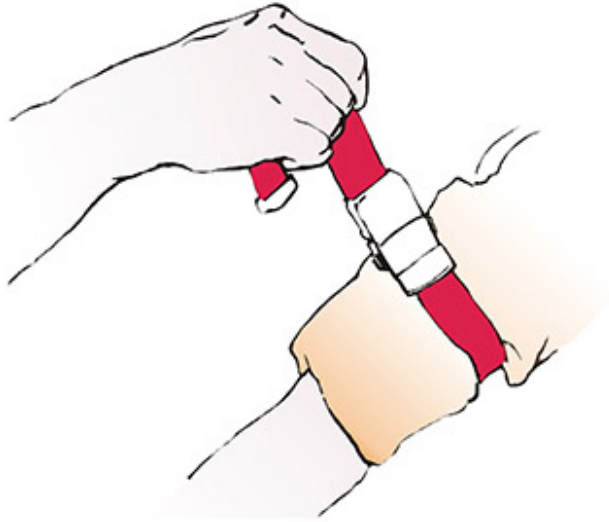


Tighten the tourniquet.

11



Put on the right glove.


12

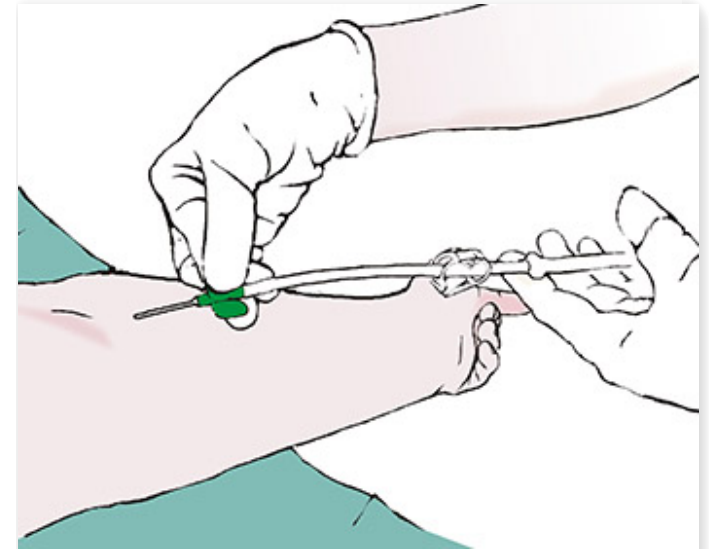

Place the cannulas into the veins.


Matoset dialysis connection set - application way

13

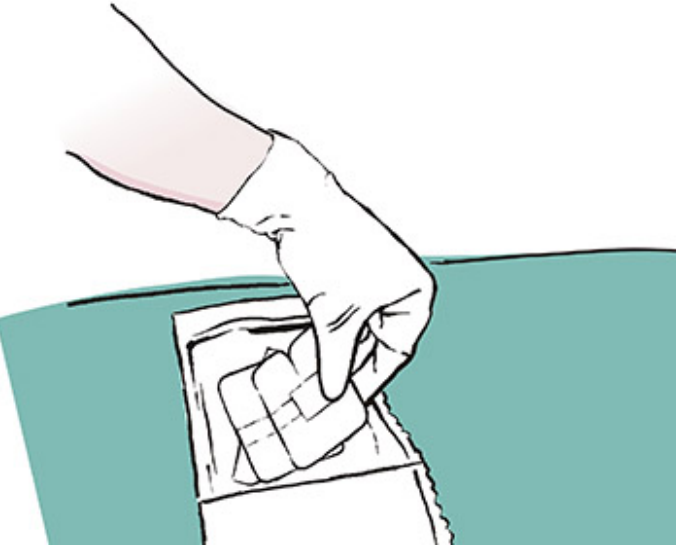


After inserting every cannula fix them with special dressings for fixing cannulas (also in the set).

14

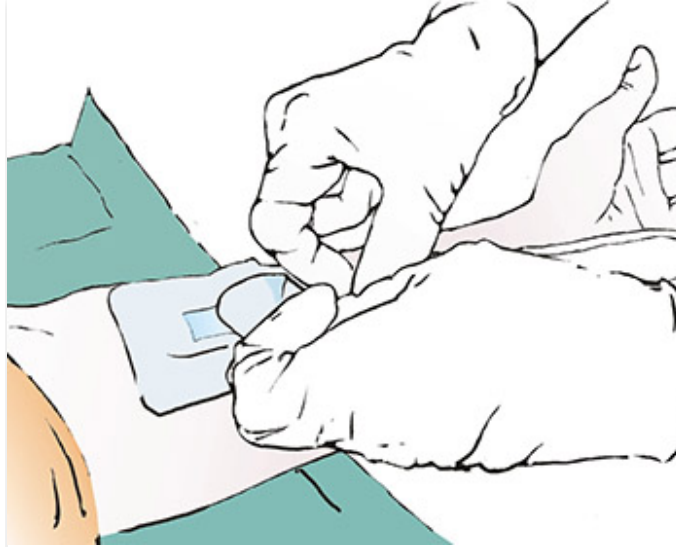


15

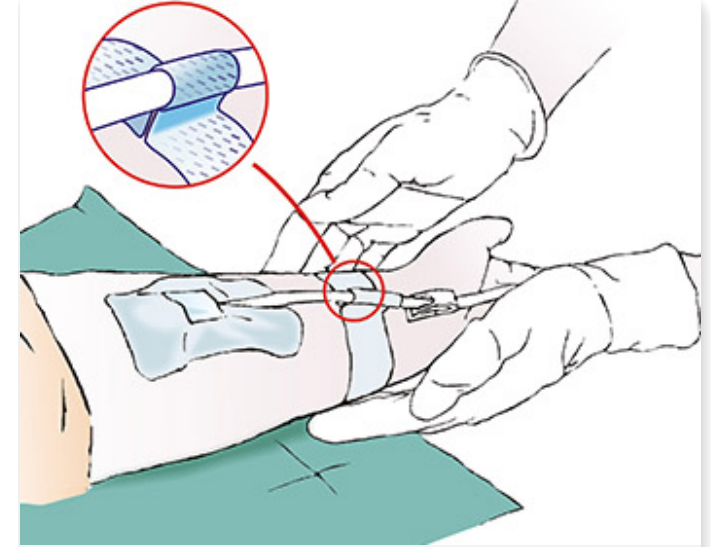

At the end secure cannula drains with the nonwoven adhesive tapes.

16

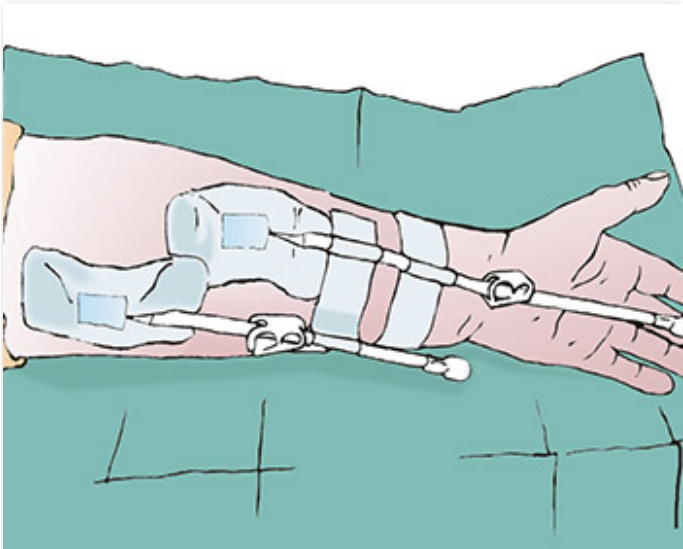

Now the patient is connected to the "artificial kidney".
Dialysis takes 3-4 hours.