


1

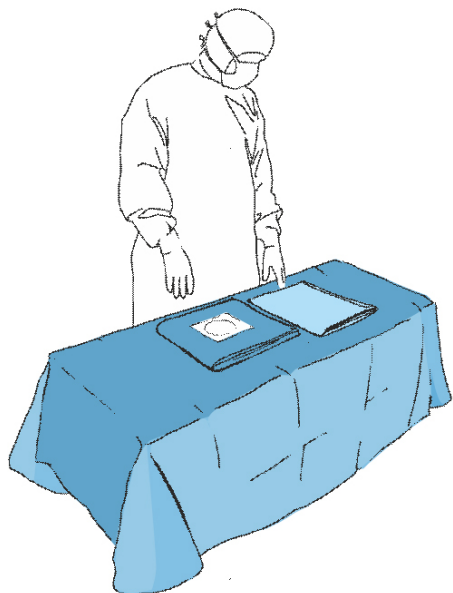


Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Spread the outer drape on the instrument table and place the individual elements of the set on it.

2



Before starting to drape the patient (determination of the surgical field), pay attention to the direction in which the drape is to be unfolded with the help of the pictograms on the drape.


3

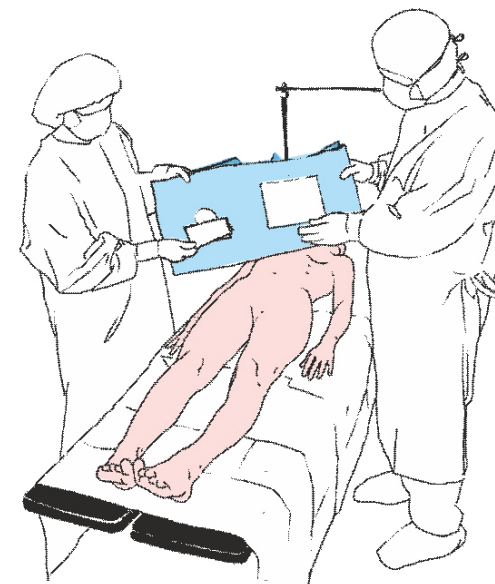

Remove the release paper from the chosen opening in the main drape (the second opening remains uncovered).


Unfolding of Matodrape angiography set

4

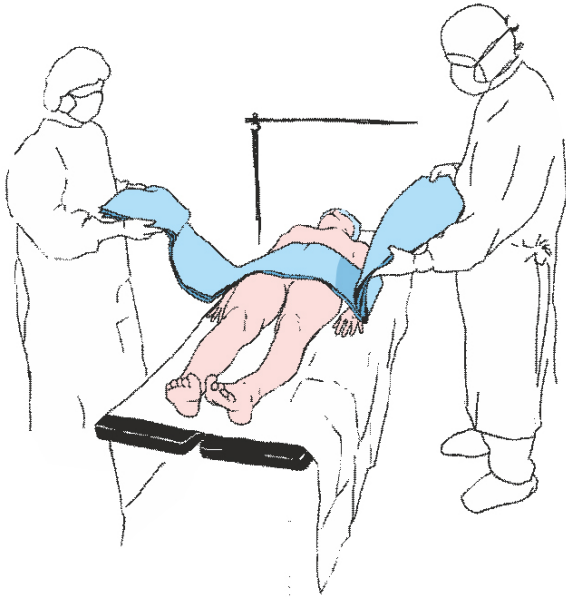

Stick the drape thus defining the operational field. Then press in the place where the adhesive is applied to increase the adhesion of the adhesive tape to the patient's skin.

5

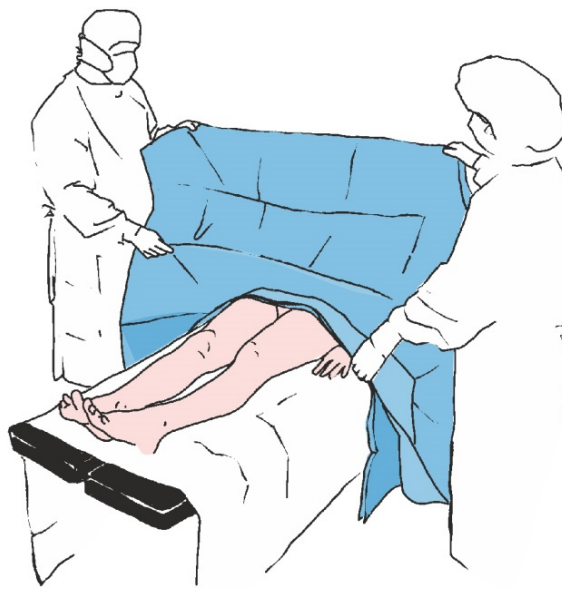


Unfold the sides of the drape so that they hang down freely from the operating table and unfold the upper part of the drape towards the patient's head in the direction of the arrows (towards the anaesthesia stand).

6




Unfold the lower part of the drape towards the lower end of the operating table covering the patient's lower limbs.

7

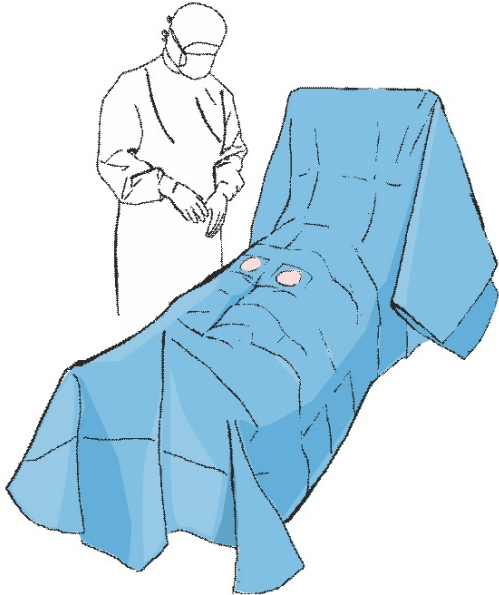

A smaller drape is placed on the anaesthesia screen separating the patient's head from the operating field.

8

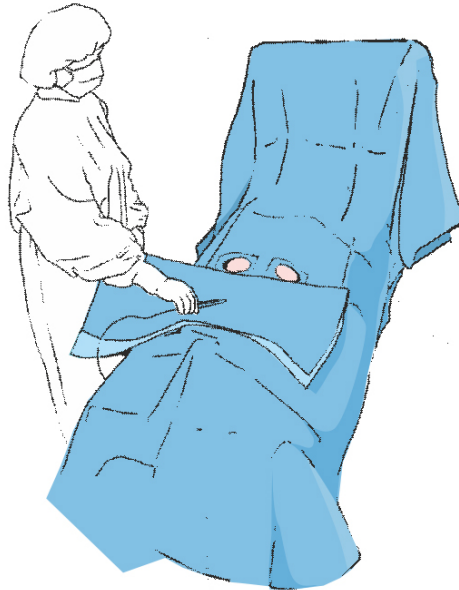

Finally, place the second smaller drape on the patient.