



1

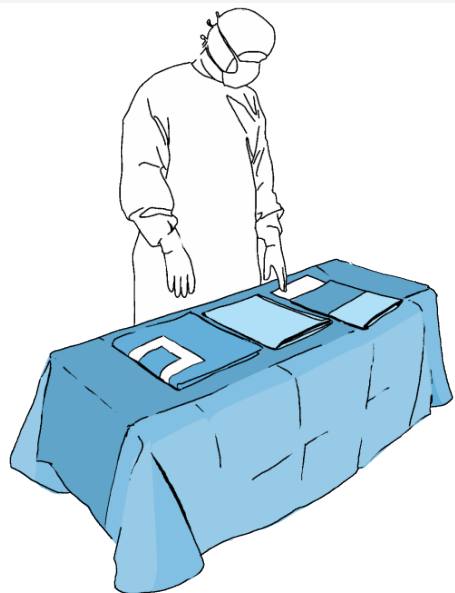


Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Unfold the outer drape on the instrument table and place set elements on it.

2

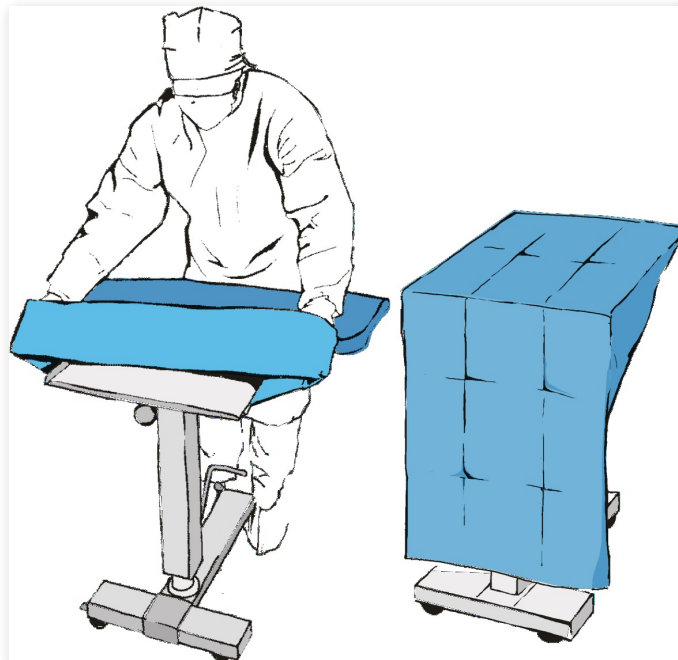

Apply the Mayo table cover.

3

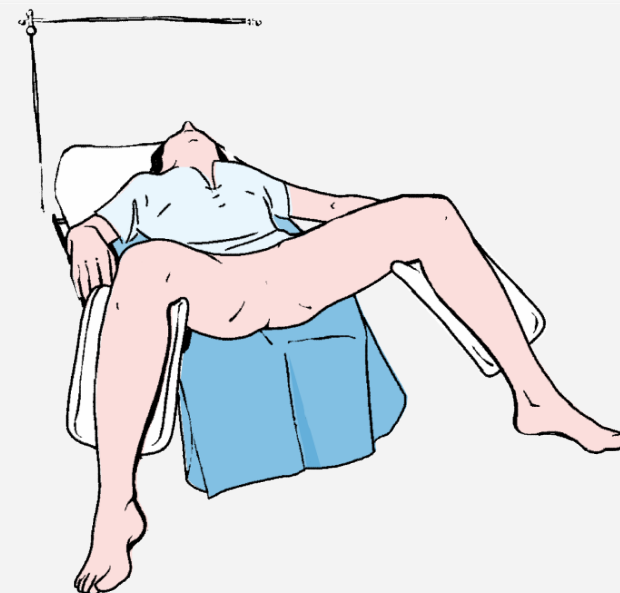

Place the smaller drape under the patient's buttocks, who is in a gynaecological position.

Unfolding of Matodrape gynaecology set

4




Apply the covers on the patient's legs.

5



Before draping the patient (delimiting the surgical field), pay attention to the direction in which the drape is to be unfolded with the help of the pictograms.


6

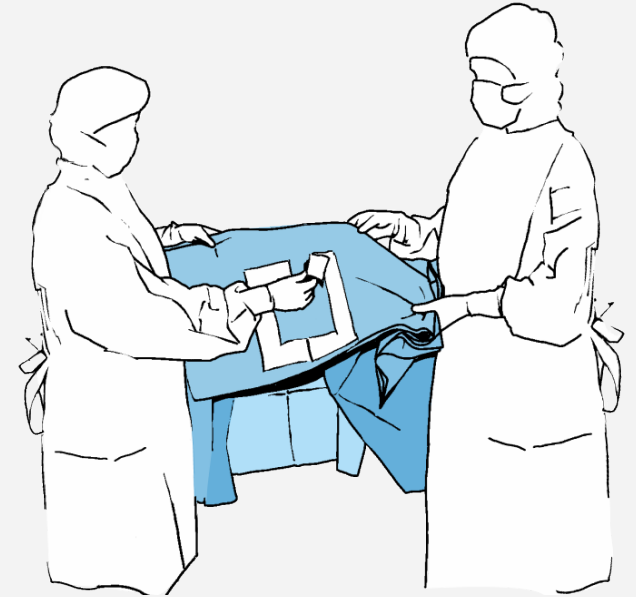

Remove the release paper from the main drape.


Unfolding of Matodrape gynaecology set

7

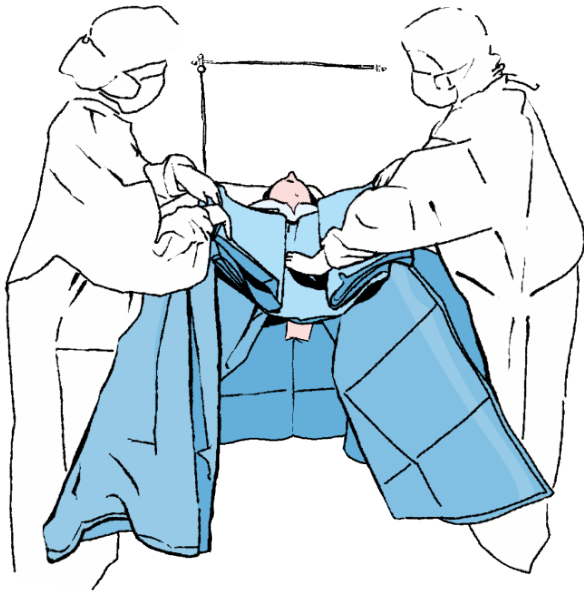

Adhere the drape to determine the borders of the surgical field. Then press the adhered area of the drape to increase the adhesion of the adhesive tape to the patient's skin.

8

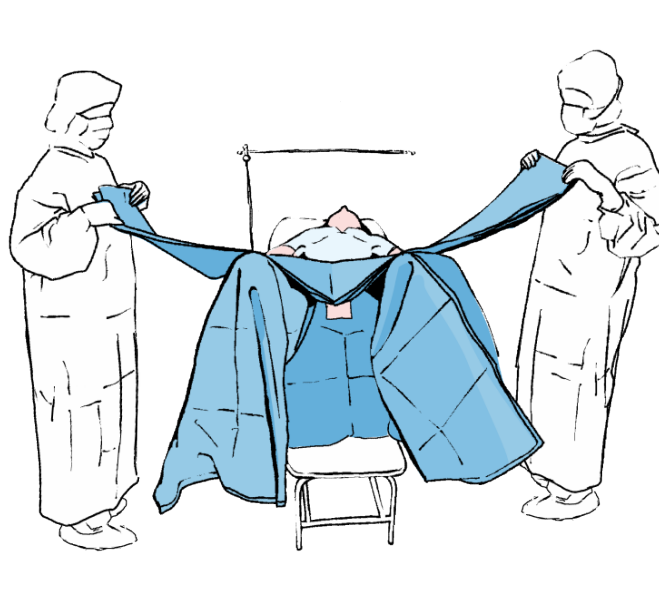

Unfold the sides of the drape so that they hang freely from the operating table, and then unfold the top of the drape toward the patient's head (towards the anaesthesia stand).

9



Pass the drape over the frame to create an anaesthesia screen. Unfold the lower part of the drape towards the lower end of the operating table, partially between the patient's legs.