



1

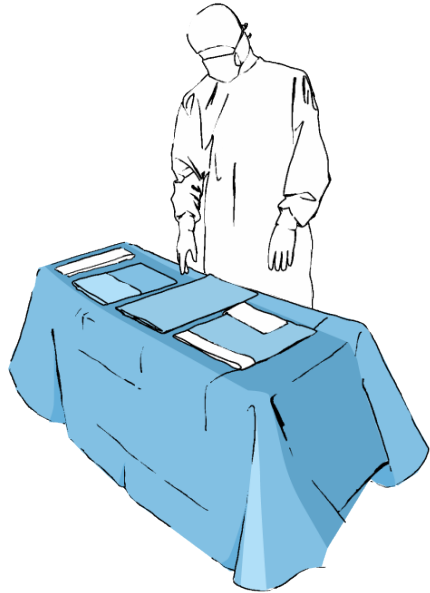


Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Unfold the outer drape on the instrument table and place set elements on it.

2

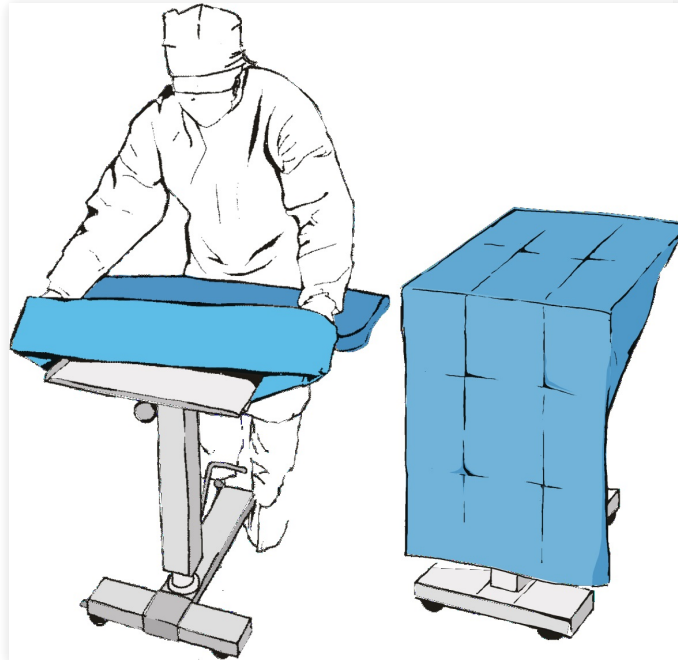

Apply the Mayo table cover.


3

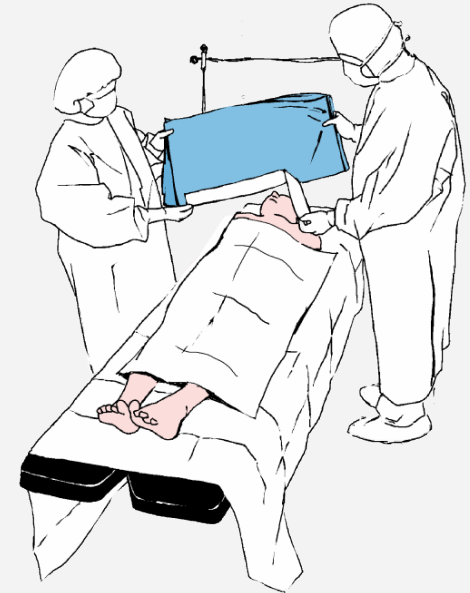

Remove the release paper from the bottom drape (only with adhesive tape).


# Unfolding of Matodrape strumectomy set

4

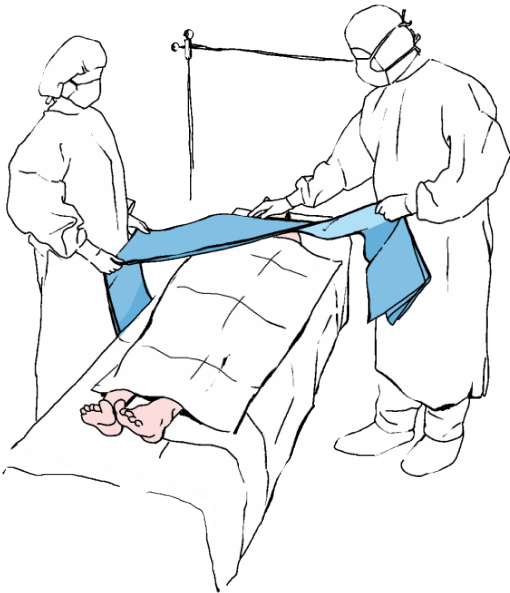


By unfolding the drape to both sides stick the drape around the top of the chest to determine the bottom edge of the surgical field.

5

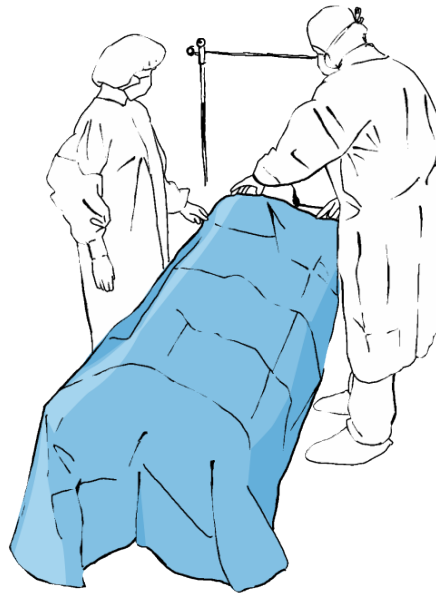

Unfold the lower part of the drape towards the lower end of the operating table to cover the lower limbs of the patient.

6

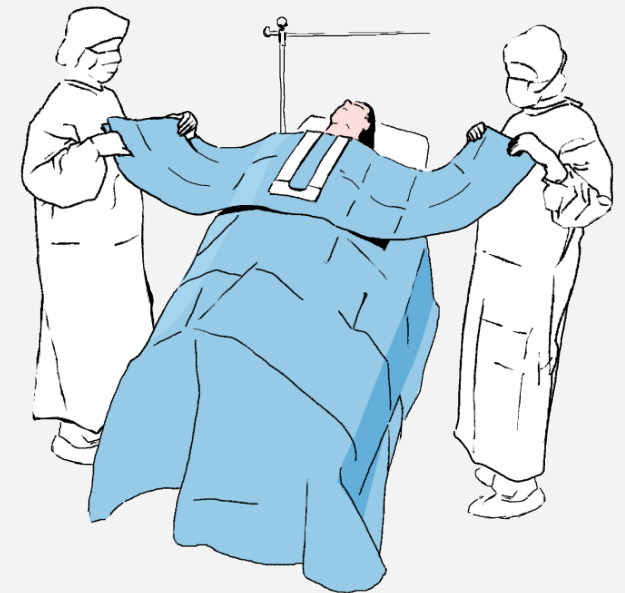

Place the drape with the "U"-opening on the bottom drape so that the adhesive tapes are on top.


# Unfolding of Matodrape strumectomy set

7




Remove the release paper.

8

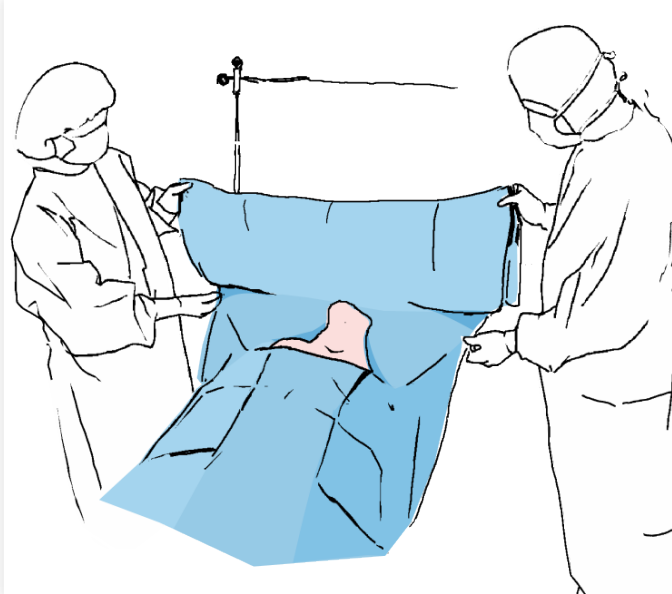

Adhere the drape to avoid wrinkles and to determine the upper and side borders of the surgical field. Then press the adhered area of the drape to increase the adhesion of the adhesive tape to the patient's skin. Unfold the drape in the direction of the arrows (towards the anaesthesia stand).

9

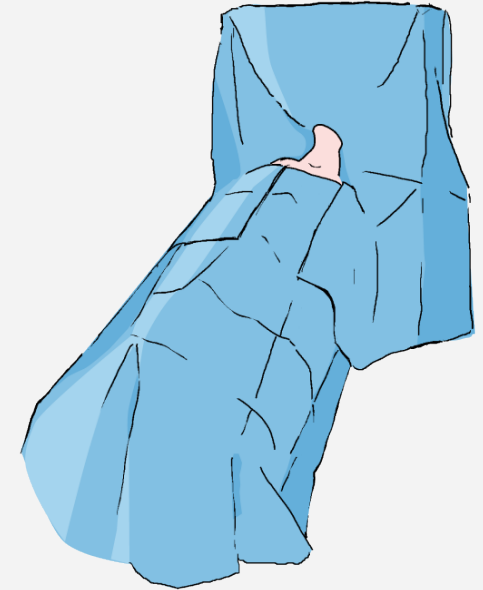

Pass the drape over the frame, stick it to the stands, to create an anaesthesia screen.