



1

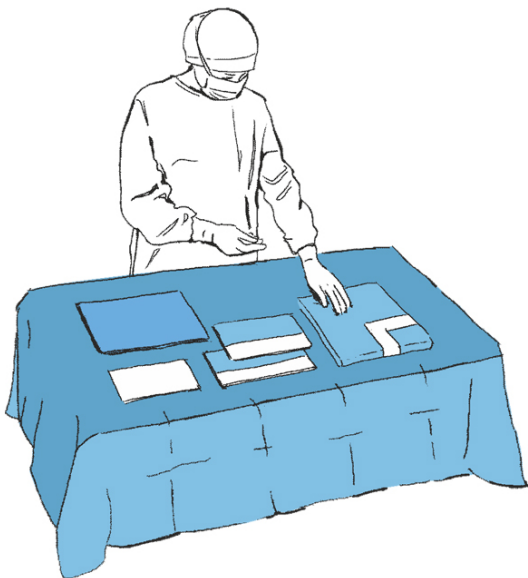

Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Unfold the outer drape on the instrument table and place set elements on it.

2

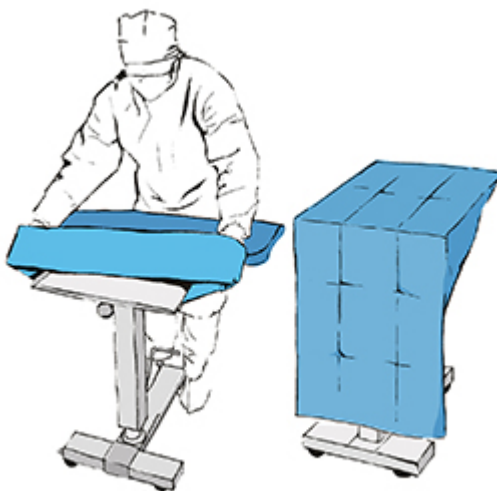

Apply the Mayo table cover.


3




Before draping the patient (delimiting the surgical field), pay attention to the direction in which the drape is to be unfolded with the help of the pictograms.

Unfolding of Matodrape thoracotomy set

4

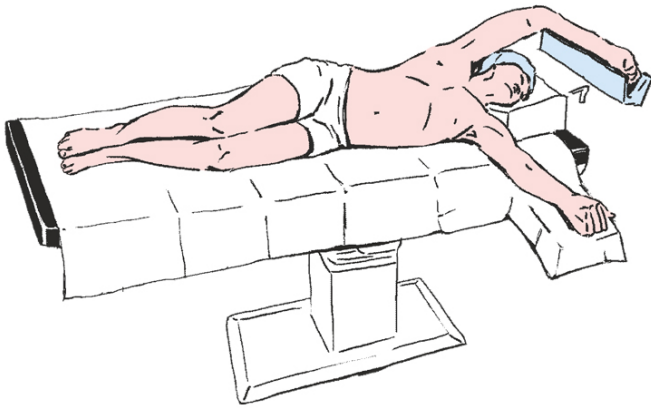


5

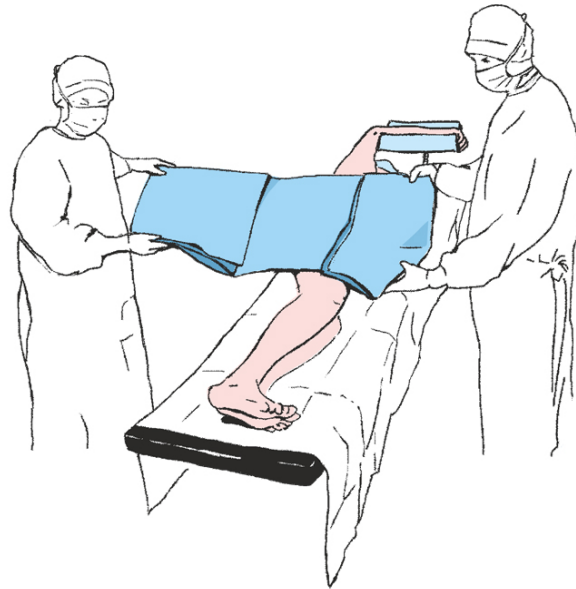

Place the drape with the "U" cut out above the patient.


6

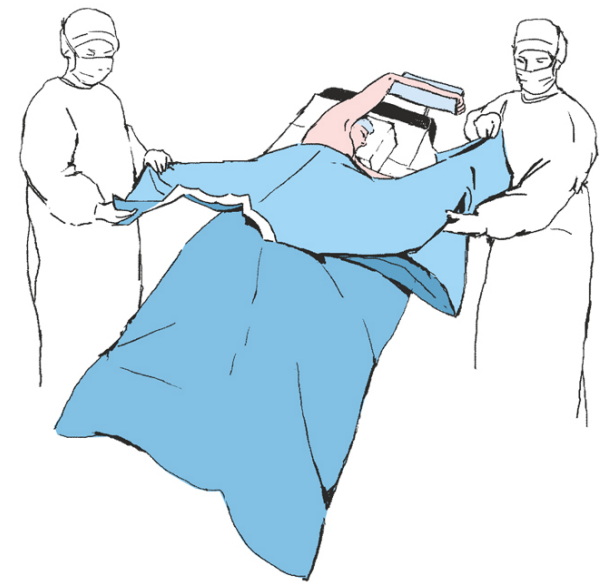

Unfold the lower part of the drape towards the lower end of the operating table, thus covering the patient's lower limbs.


Unfolding of Matodrape thoracotomy set

7

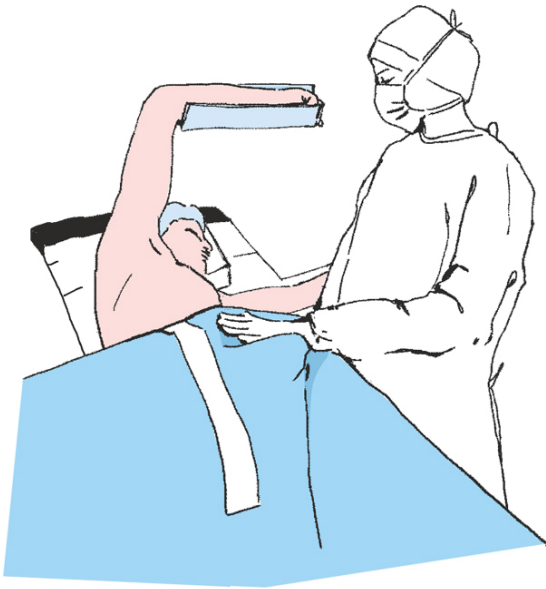


Remove the release paper from the shorter section of the adhesive tape and stick it to the patient's lower chest.

8

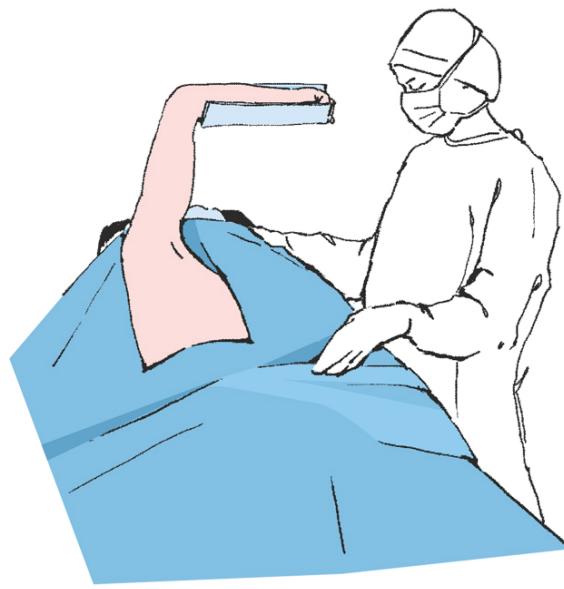

Remove the remaining release papers and stick the drape to the sides of the chest carefully towards the patient's head, thus determining the lower and lateral limits of the operating field. Press in the adhered place to increase the adhesion to the patient's skin.

9

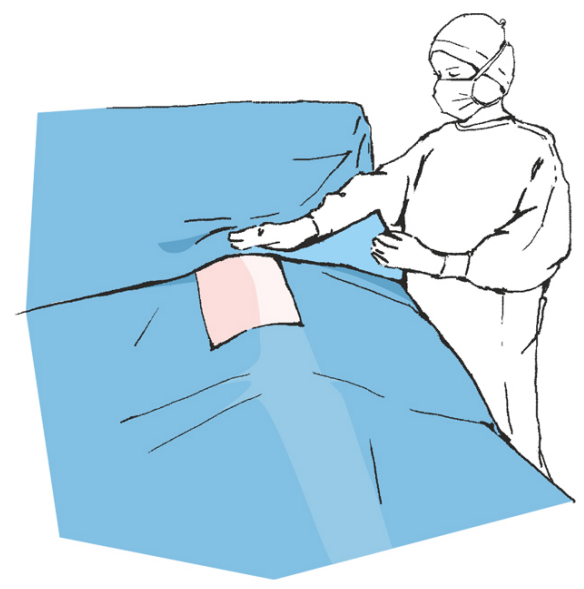

Remove the release paper from the upper drape and stick it to the upper part of the chest. Press in the adhered place to increase the adhesion to the patient's skin. Unfold the drape in the direction of the arrows over the frame, thus creating an anaesthesia screen.

10

