



1

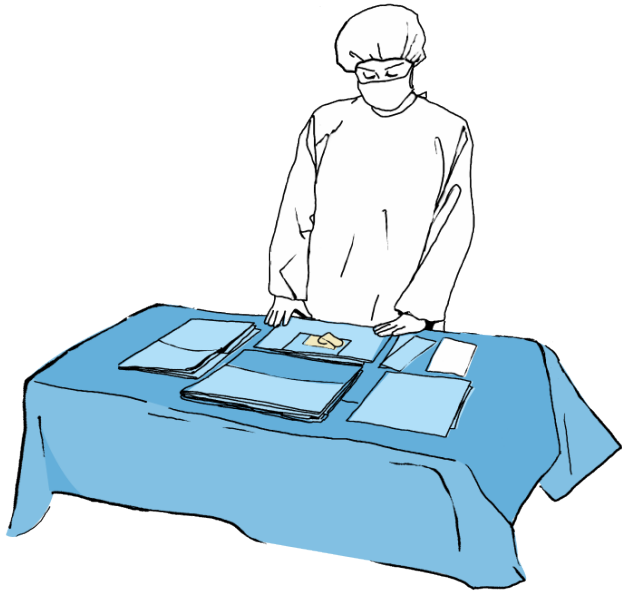


Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Unfold the outer drape on the instrument table and place set elements on it.

2

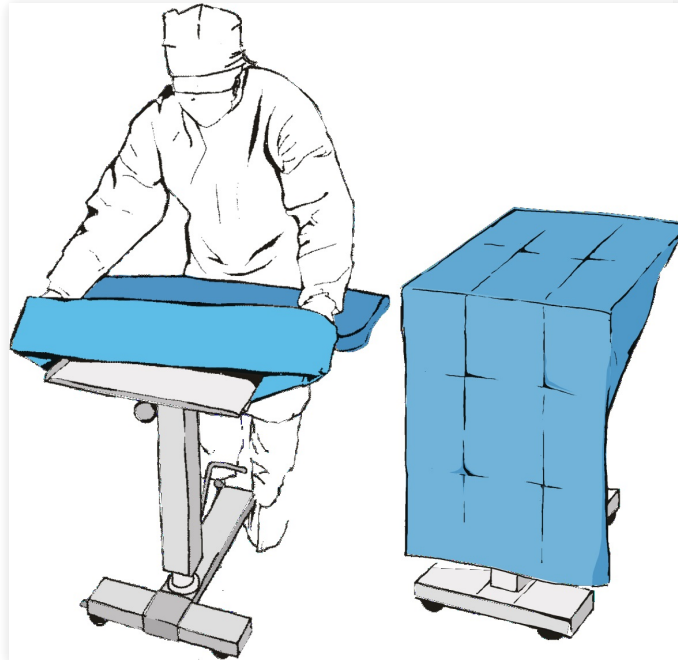

Apply the Mayo table cover.


3

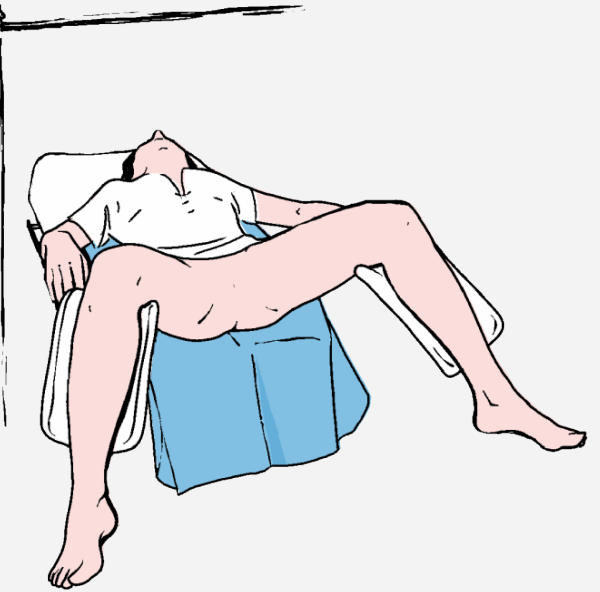

Place the smaller drape under the patient's buttocks, which is in a urological position.

Unfolding of Matodrape urology set (TUR)

4

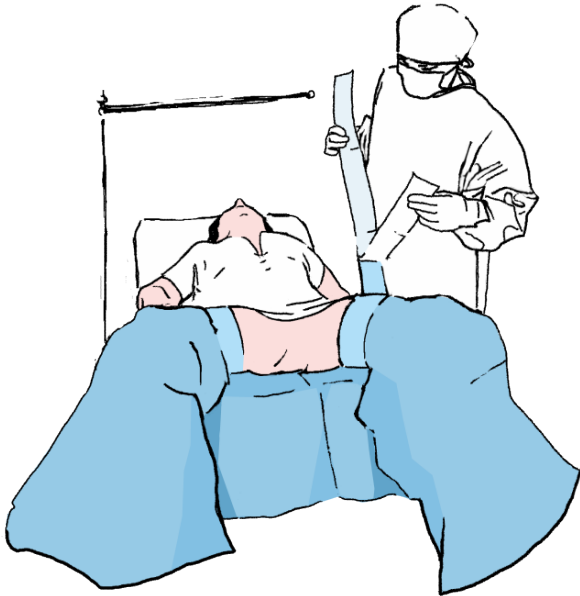


Apply the covers on the patient's legs and fix them with medical tapes.

5



Before draping the patient (delimiting the surgical field), pay attention to the direction in which the drape is to be unfolded with the help of the pictograms.


6

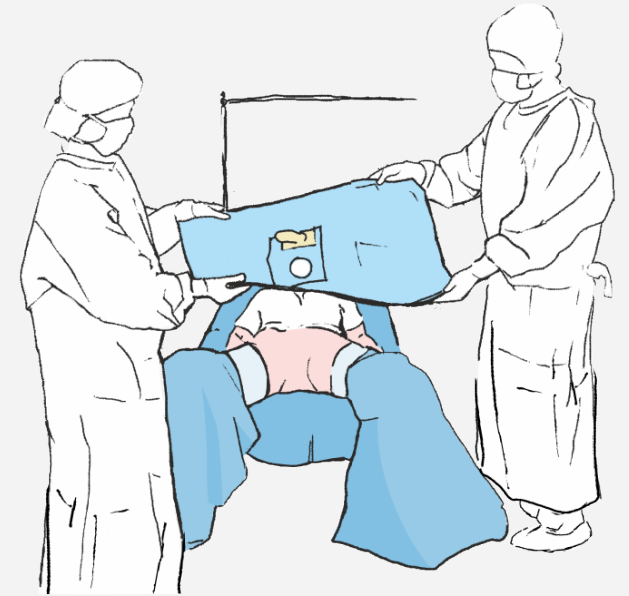

Partially unfold the main drape over the patient and adjust the openings in the appropriate areas of the surgical field and place the drape on the patient to determine the surgical field.

Unfolding of Matodrape urology set (TUR)

7

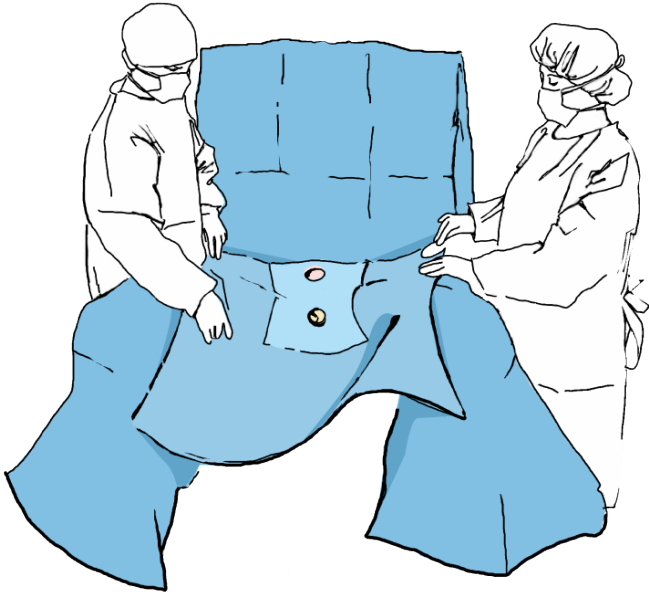

Unfold the sides of the drape so that they hang freely from the operating table, and then unfold the top of the drape toward the patient's head (toward the anaesthesia stand).
Pass the drape over the frame to create an anaesthesia screen. Open the lower part of the drape towards the lower end of the operating table, partially between the patient's legs.

8

