



1

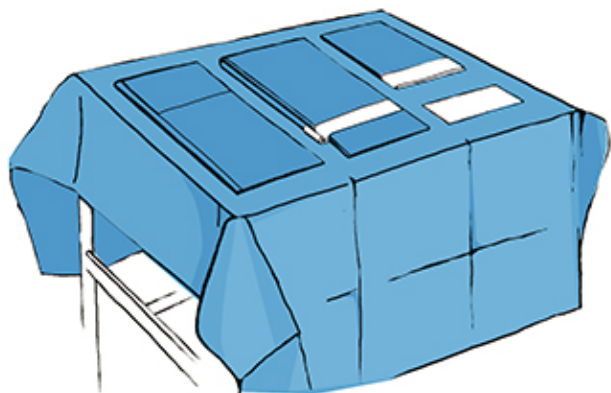

Having opened the set put it on the instrumental table. Use the outer drape to secure the table and remaining elements. Put the elements on the table paying attention to the pictograms on the drape (they show where to place each element).

2

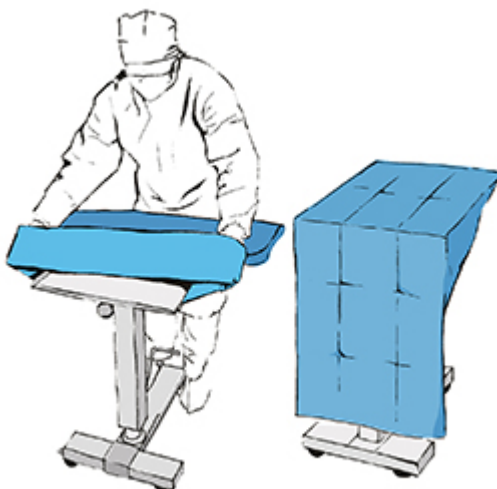

Apply the Mayo table cover (in form of a bag with a cuff).


3



Remove the release paper from the lower drape marked with a foot pictogram.


# Unfolding of Matodrape universal set

4

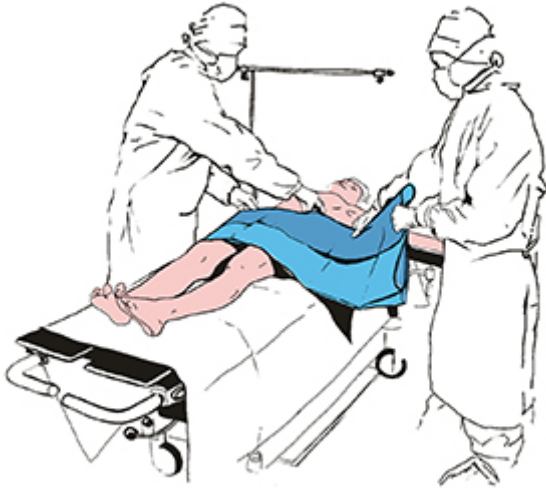


Fix the drape on the patient's body delimiting the OP-field line. Unfold the remaining part of the drape onto the patient's legs.

5

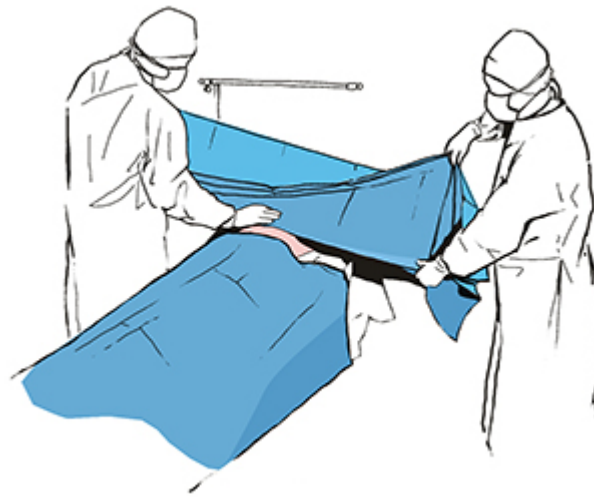

The upper OP-field line is to be secured by fixing the drape from the patient's head side (drape marked with a head pictogram). Having done that unfold the drape and fix it in the anaesthesia screen using the adhesive tapes in the corners of the drape..

6

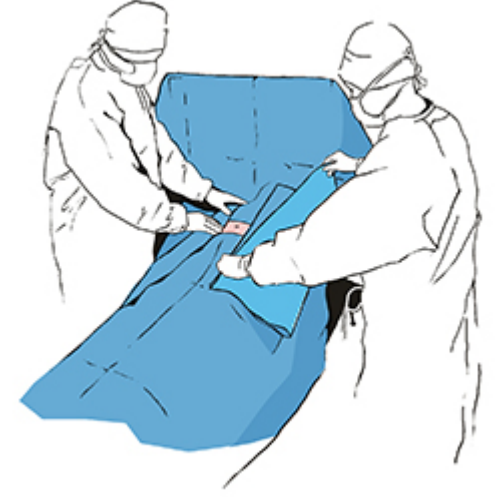

Fix the side drapes to complete the delimiting of the OP-field.