



1

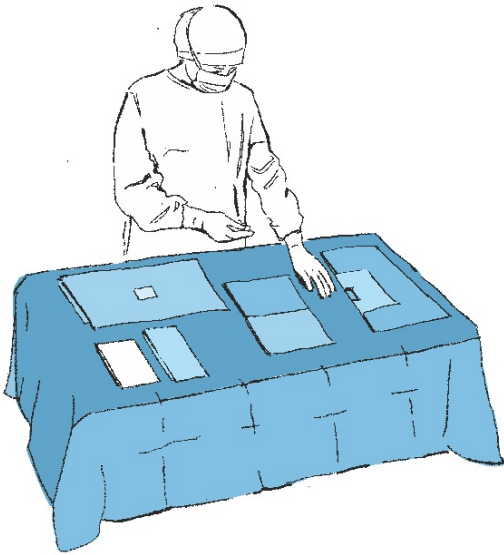


Open the outer packaging in accordance with the principles of asepsis and place the set on the instrument table. Unfold the outer drape on the instrument table and place set elements on it.

2

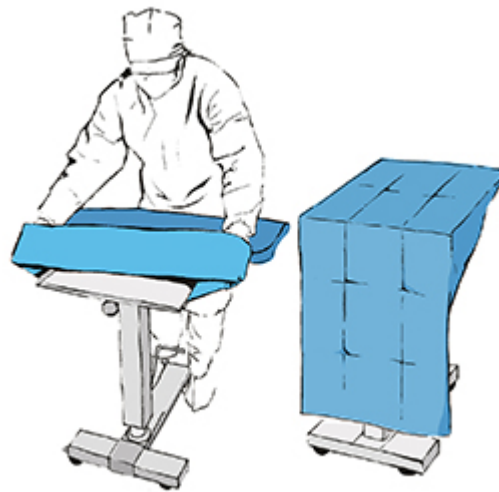

Apply the Mayo table cover.


3

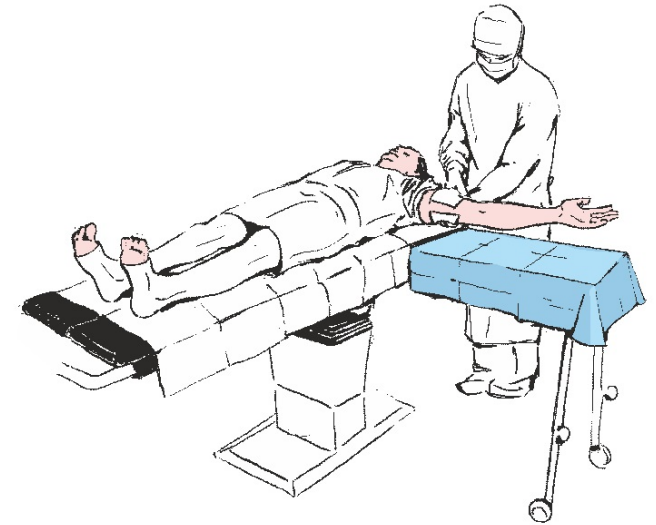

Lift the upper limb that is subject to the medical procedure, using the towels included in the set.


# Unfolding of Matodrape upper limb set

4

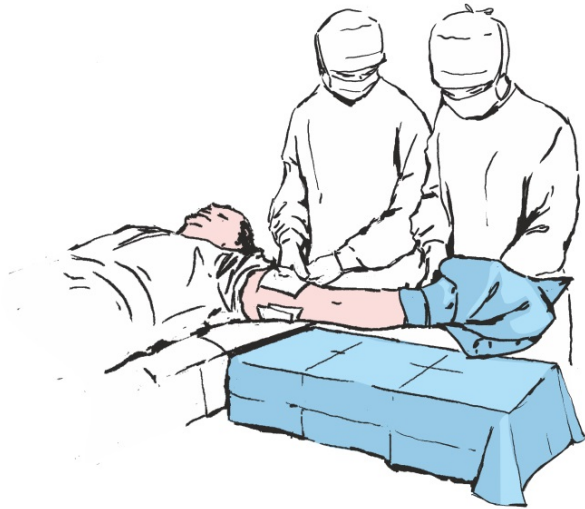

Slide the cover on the patient's upper limb.

5

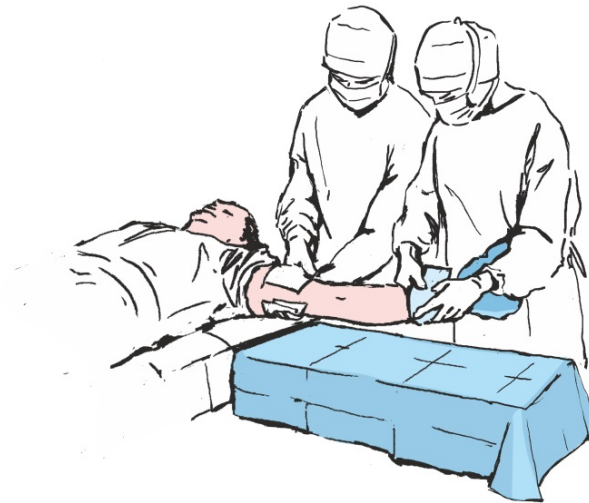

Fix the cover at the height of the patient's forearm using the fixing tapes provided.


6



Before starting to drape the patient (determination of the surgical field), pay attention to the direction in which the drape is unfolded with the help of the pictograms on the drape.


# Unfolding of Matodrape upper limb set

7

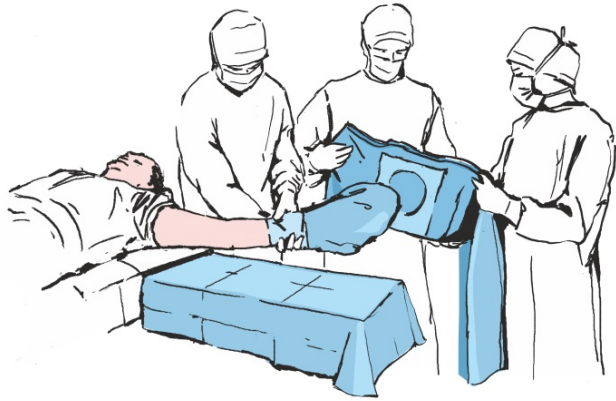


Pass the patient's upper limb through the elastic opening in the main drape.

8

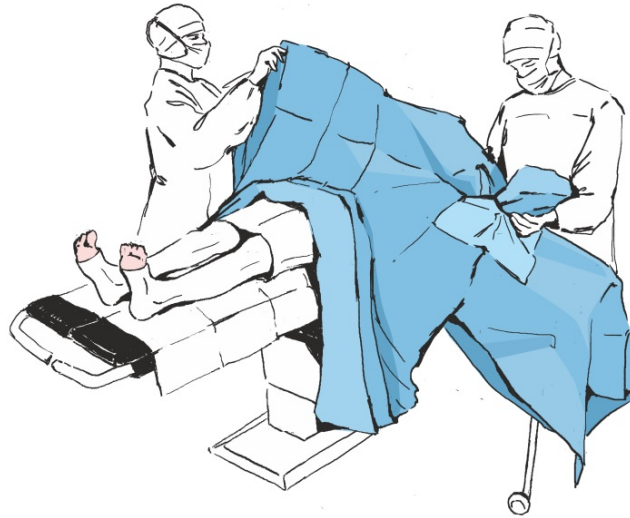

Cover the limb stand with a side drape permanently attached to the main drape.

9

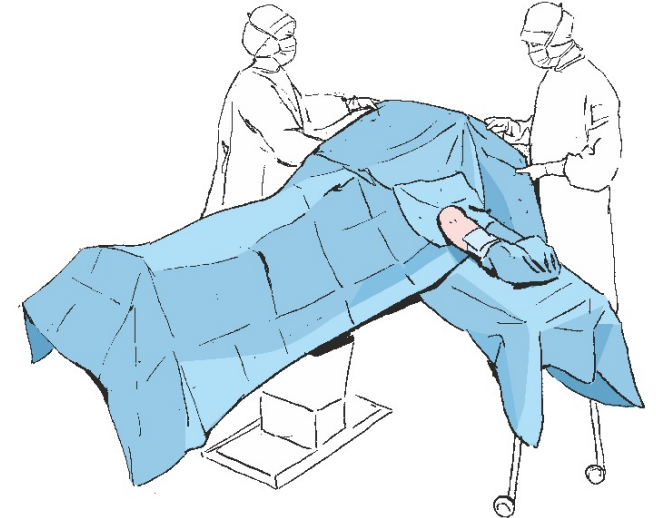

Unfold the lower part of the drape towards the lower edge of the operating table, thus protecting the patient's lower limbs and then unfold the drape towards the anaesthesia screen. Pass the drape over the frame to create an anaesthesia screen.

10

